One-press Serter

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RELEASED

Introduction

The one-pressserter (MMT-7512) for Enlite™ glucose sensors (MMT-7008A, MMT-7008B) is used as an aid for inserting the Enlite sensor.

Potential risks related to serter use

General risks with serter use may include skin infection around the area where the serter is used.

Indications for use

The serter is used as an aid for inserting the Enlite sensor. It is indicated for single-patient use.

Contraindications

Do not use the serter on products other than the Enlite sensor. Medtronic cannot guarantee the safety or efficacy of this product if used with other products.

General warnings

Read this entire user guide before attempting to insert the sensor. The serter does not work the same as other Medtronic insertion devices. Failure to follow directions may result in improper insertion, pain, or injury.

Contact the 24 Hour HelpLine or your local representative, if you have questions or concerns.

Do not make therapy decisions based on sensor glucose values because sensor glucose and blood glucose values may differ. If your sensor glucose reading is low or high, or if you feel symptoms of low or high glucose, do the following prior to making therapy decisions. Confirm your blood glucose with your BG meter using a fingerstick blood sample.

A retractable needle is attached to the sensor and minimal blood splatter may occur. If you are a healthcare professional or caregiver, wrap sterile gauze around the sensor to minimize contact with blood. Keep as much distance as possible between you and the patient when removing the needle.

Keep the needle housing within sight at all times to avoid an accidental needlestick or puncture.

Always inspect the sensor packaging for damage before use. Sensors are sterile and non-pyrogenic, unless the package has been opened or damaged. Do not use the sensor if the sterile package has been opened or damaged. Use of an unsterile sensor can cause site infection.

Watch for bleeding at the insertion site (under, around, or on top of the sensor).

If bleeding occurs, do the following:

1. Apply steady pressure using sterile gauze or a clean cloth, placed on top of the sensor for up to three minutes. The use of unsterile gauze can cause site infection.
2. If bleeding stops, connect the transmitter to the sensor.

If bleeding does not stop, do not connect the transmitter (or recorder) to the sensor because blood can get into the transmitter connector, and could damage the device.

If bleeding continues, do the following:

1. Remove the sensor and continue to apply steady pressure until the bleeding stops. Discard the sensor in a sharps container.
2. Check the site for redness, bleeding, irritation, pain, tenderness, or inflammation. Treat based on instructions from your healthcare professional.
3. Insert a new sensor in a different location.

Make sure the sensor is securely placed in the serter to avoid improper insertion, pain, or minor injury. Keep the serter away from children. This product contains small parts and may pose a choking hazard.

Refer to the *enlite user guide* for additional sensor related warnings.

General precautions

Wash your hands with soap and water before inserting the sensor to help prevent site infection.

If you are a healthcare professional or caregiver, wear gloves when inserting the sensor into someone other than yourself to avoid contact with patient blood.

Do not insert the sensor through tape. Inserting the sensor through tape may cause improper sensor insertion and function.

Only use alcohol to prepare the insertion site, to ensure that residue is not left on the skin.

Rotate the sensor insertion sites so that they do not become overused.

Discard used sensors and needle housings in a sharps container after each use to avoid accidental needlestick or puncture.

Do not clean, resterilize, or try to extract the needle from the needle housing. An accidental needlestick or puncture may occur.

Do not reuse sensors. Reuse of a sensor may cause damage to the sensor surface and lead to inaccurate glucose values, site irritation, or infection.

Always make sure the sensor is firmly attached so that it does not come out of your body. Factors that may increase the likelihood of the sensor falling off include increased physical activity (particularly for younger patients) and improper taping technique.

Where to insert the sensor

CAUTION: Make sure the sensor site is at least 2.5 cm (1 inch) away from the insulin pump infusion site or manual injection site. When replacing the sensor, select a new site that is at least 2.5 cm (1 inch) away from the previous site to allow the area to heal. Any scarring or hardening of the tissue can cause inaccurate insulin delivery or sensor performance.

CAUTION: Avoid the 5.0 cm (2 inch) area around the navel to help ensure a comfortable insertion site and to help with sensor adhesion.

Choose an insertion site that has an adequate amount of subcutaneous fat. The Enlite sensor has been approved for use on the abdomen, the lower back, or the upper buttock areas. For a diagram of the area, see *Inserting the sensor, step 2*.

Do not insert the sensor in muscle or areas constrained by clothing or accessories, areas with tough skin or scar tissue, sites subjected to rigorous movement during exercise, or in sites under a belt or on the waistline for best sensor performance and to avoid accidental sensor removal.

Removing the sensor

Always refer to the *enlite user guide* for instructions on how to remove the sensor.

Storage and handling

Store the senter in the released position, at room temperature, to maintain optimum product performance and life.

Specifications

Approximate Dimensions	Approximate Weight
7.85 x 6.91 x 5.77 centimeters (3.09 x 2.72 x 2.27 inches)	112.5 grams (3.97 ounces)

Serter life of use

The serter has a minimum service life of one year.

Serter maintenance

The serter is intended for single-patient use. The serter needs to be cleaned after every use or whenever there is debris, blood, or other contaminants.

Cleaning

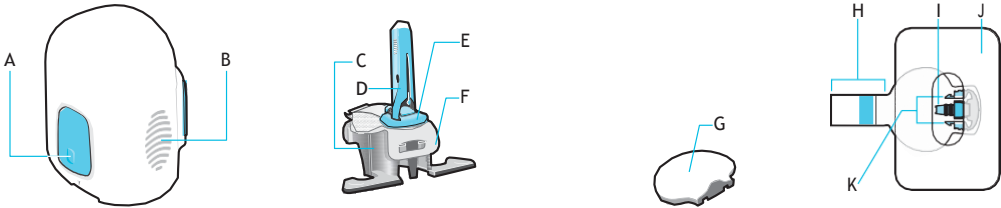
To clean the serter, you will need the following materials: mild liquid soap, a soft-bristled toddler toothbrush, and a container.

To clean the serter:

1. Prepare a mild liquid soap solution using 5 milliliters (1 teaspoon) of mild liquid soap per 3.8 liters (1 gallon, which is equal to 16 cups) of tap water.
2. Rinse the serter under running tap water at room temperature for at least one minute. Continue rinsing until the serter is visibly clean. Make sure all hard-to-reach areas are rinsed completely.
3. When rinsing the serter, press and release the mechanism on the underside of the serter to ensure that the entire device is completely rinsed with water.
4. Submerge the serter in the mild liquid soap solution and soak it for at least 10 minutes.
5. Brush the entire surface of the serter using a soft-bristled toddler toothbrush until visibly clean. Pay close attention to brushing the hard-to-reach areas.
6. Rinse the serter under running tap water at room temperature for at least one minute, and until all visible liquid soap is gone.
7. Shake off any excess water or moisture and place the serter upright on a clean, dry, cloth to air dry.

©2015, Medtronic MiniMed, Inc. All rights reserved. Enlite™ is a trademark of Medtronic MiniMed, Inc.

Components

Enlite sensor system

One-press sarter
A. Bump on both buttons
B. Thumbprint marking

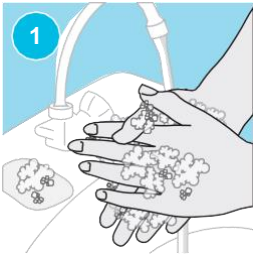
C. Pedestal
D. Needle housing
E. Sensor
F. Clear liner

G. Transmitter (or
recorder)


H. Adhesive tab
I. Sensor connector
J. Enlite overtape
K. Sensor snaps

Inserting the sensor


WARNING: Wear gloves when inserting the sensor into someone other than yourself to avoid contact with patient blood. Minimal bleeding may occur. Contact with patient blood can cause infection.

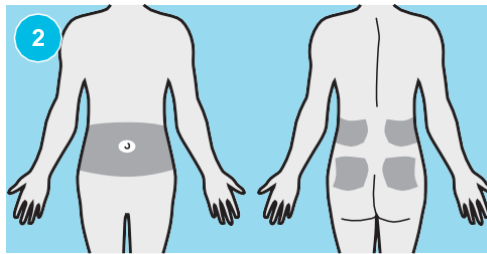1. Wash your hands.

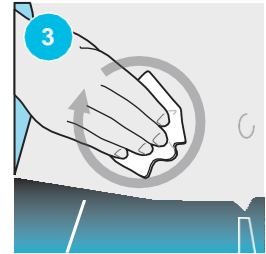
2. Choose an insertion site that has an adequate amount of fat, either on the abdomen, the lower back, or the upper buttock areas. Shaded areas show the best sites for sensor insertion.


3. Clean the insertion site with alcohol. Let the area air dry.

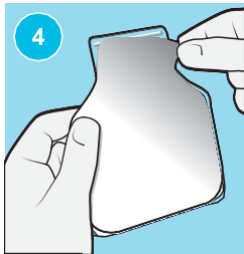
RELEASED

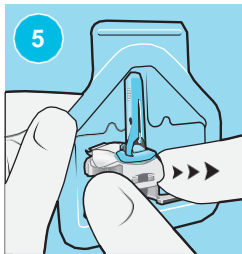
4. Open the sensor package.

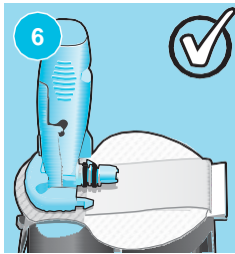
5. Remove the pedestal from the package. Place the pedestal on a flat surface.

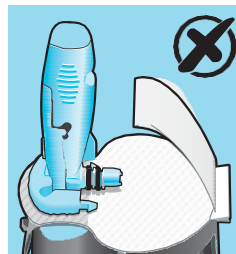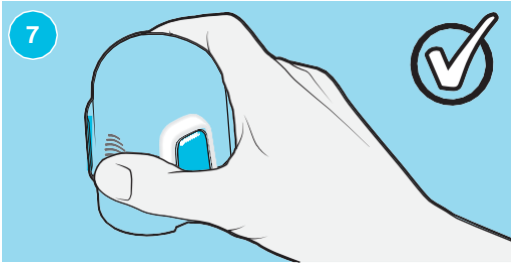
6. Make sure that the Enlite sensor's adhesive tab is tucked under the sensor connector and sensor snaps.



7. Holding serter Correctly

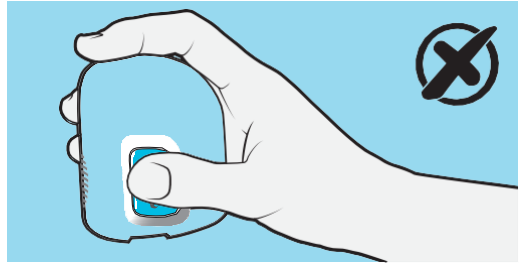Place your thumb on the thumbprint marking to hold the serter without touching the buttons.


Holding serter Incorrectly

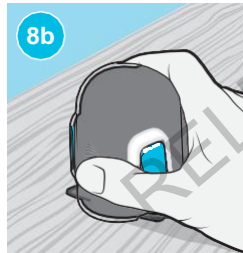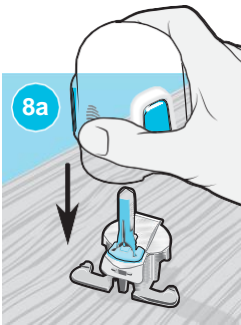
Your fingers should not be touching the buttons.

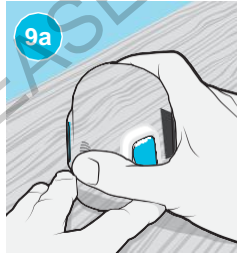
8a–8b. Grip the serter, placing your thumb on the thumbprint marking, **without holding the buttons**. Carefully push the serter down onto the pedestal until the base of the serter sits flat on the table and you hear a click.

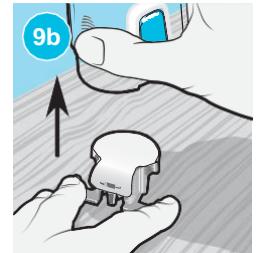9a. To detach the serter from the pedestal, using one hand, place your thumb on the thumbprint marking to grip the serter **without holding the buttons**.



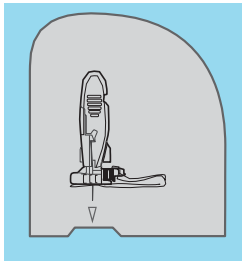
9b. With your other hand, place two fingers on the pedestal arms and slowly pull the serter straight up. Do not detach the pedestal from the serter in midair, as this might damage the sensor.

Note: The arrow on the side of theserter aligns with the needle inside theserter.

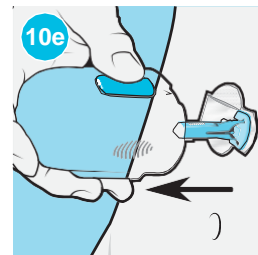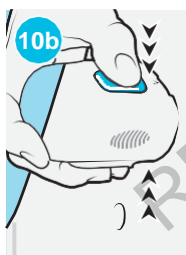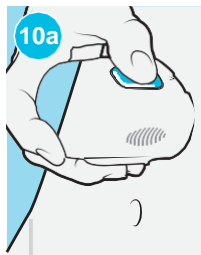WARNING: Never point a loadedserter toward any body part where insertion is not desired. An accidental button-push may cause the needle to inject the sensor in an undesired location, causing minor injury.

10a. Hold theserter steady against your cleaned insertion site, without pushing theserter too deep into your skin.

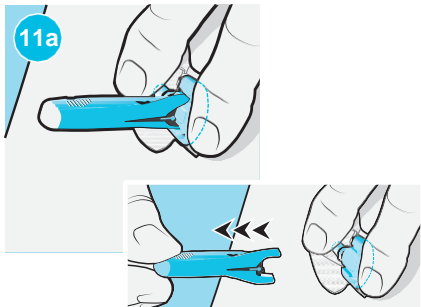
10b-10c. Press and release the bump on both buttons at the same time, while holding theserter flat against your body.

10d. Continue holding theserter flat against your body to allow the adhesive to stick to your skin.

10e. Slowly lift theserter away from your body, making sure that the buttons are not pressed.

Note: Failing to hold theserter securely flat against your body may allow theserter to spring back after pressing the buttons, and result in improper insertion of the sensor.


If you inserted the sensor into yourself, complete step 11a. If you are a healthcare professional or caregiver who inserted the sensor into a patient, complete step 11b.

Patient:

11a. Gently hold the base of the sensor against skin. Hold the needle housing **at the top** and slowly pull straight out, away from the sensor.

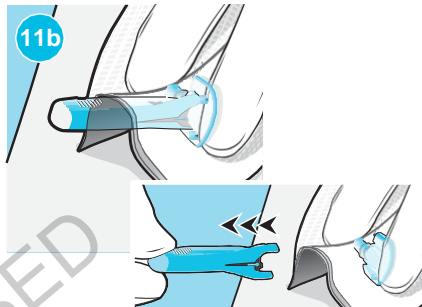OR


Healthcare professional or caregiver:


11b. Wrap sterile gauze around the sensor (as shown in image 11b). Gently hold the base of the sensor against skin. Hold the needle housing **at the top** and slowly pull straight out, away from the sensor.

WARNING: Watch for bleeding at the insertion site. If bleeding occurs under, around, or on top of the sensor, apply steady pressure using sterile gauze or a clean cloth placed on top of the sensor for up to three minutes. The use of unsterile gauze can cause an infection. If bleeding does not stop, remove the sensor and apply steady pressure until the bleeding stops.

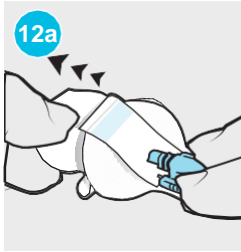Note: Medtronic adhesives are pressure-sensitive. Pressing the adhesive against the skin ensures that the sensor remains adhered to the skin throughout the wear period.

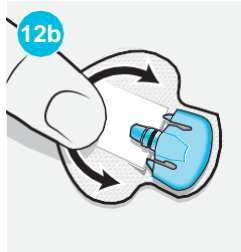
12a. Hold the sensor in place and gently remove the adhesive liner from under the adhesive pad. Do not remove the adhesive liner from the rectangular adhesive tab. This tab will be used to secure the transmitter in a later step.

12b. Firmly press the adhesive pad against the skin to make sure that the sensor remains adhered to the skin.

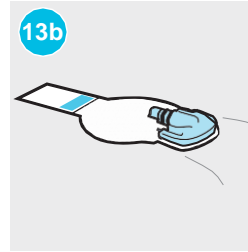
13a. Untuck the adhesive tab from under the connector.


13b. Straighten the sensor adhesive tab so that it lies flat against the skin.

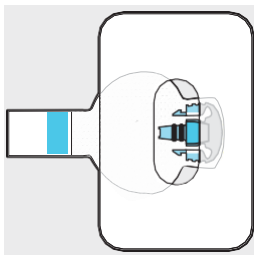Applying the overtape


This image shows an example of the Enlite overtape applied correctly. To apply the overtape, go to step 1.

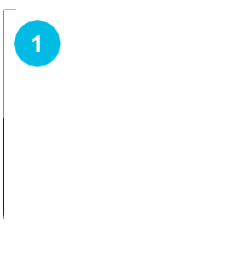
Note: The sensor body and the skin must be covered partially as shown in the image. The sensor connector and sensor snaps must be uncovered and accessible.

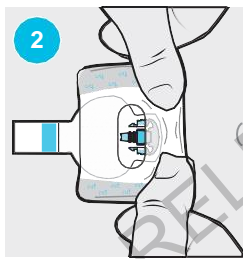
1. Remove the adhesive liner marked "1" from the overtape. Do not remove the adhesive liner marked "2" from the sides of the overtape.

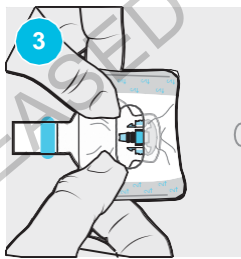2. Apply the overtape to the rounded part of the sensor and the skin, making sure that the sensor connector and sensor snaps are sticking out through the hole in the overtape.

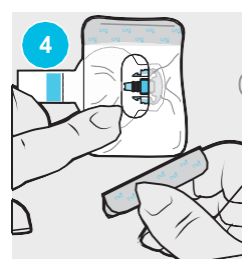
3. Stretch the overtape around the sensor connector and sensor snaps so that it lies flat against the sensor adhesive pad.


4. Remove the adhesive liner marked "2" from both sides of the overtape and smooth the overtape to the skin.

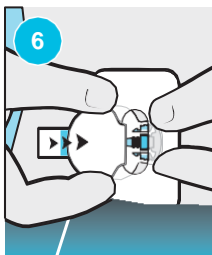
5. For details on how to enter your sensor settings into your pump or monitor, refer to the corresponding user guide.

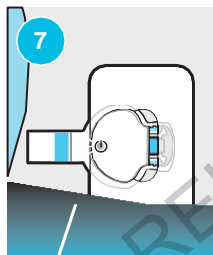6. Connect the transmitter to the sensor.

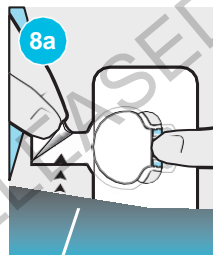
7. Wait for the green light on the transmitter to flash. If the green light on the transmitter does not flash, refer to the Troubleshooting section of your transmitter user guide.

- 8a. Remove the adhesive liner from the adhesive tab.

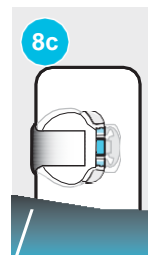- 8b-8c. Cover the transmitter with the adhesive tab. Do not pull the tab too tightly.


9. To apply a second overtape, complete Option 9a or Option 9b.

If you have skin irritation due to moisture buildup, complete Option 9a. If the transmitter catches on your clothes, complete Option 9b.

Note: Use the Enlite overtape included in the sensor box to secure the transmitter. If you run out of Enlite overtape, you may use over the counter medical tape.

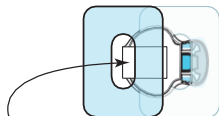9a


End of transmitter is **exposed**

Option 9a

Position the overtape so that the end of the transmitter is exposed through the opening in the tape.

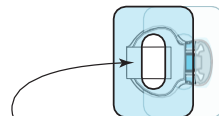9b

End of transmitter is **covered**

Option 9b

Position the opening of the overtape over the center of the transmitter. Make sure that the end of the transmitter is securely covered by the overtape.

OR

Icon table

	Consult instructions for use.		Caution
	Batch code	xxx	Humidity limitation
(1X)	One per container/package		Date of manufacture (YYYY-MM-DD)
$\begin{matrix} +xxx^{\circ}\text{C} \\ -xxx^{\circ}\text{C} \\ +xxxx^{\circ}\text{F} \\ -xxx^{\circ}\text{F} \end{matrix}$	Temperature limit		Authorized representative in the European Community.
	Catalogue or model number		Marking of European conformity: device fully complies with MDD 93/42/EEC and other directives.
	Manufacturer		

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Medtronic Ltd.
Medtronic Österreich GmbH
Medtronic B.V.